Contents: 2 Nipplettes, 2 Suction Devices,
2 Disposable Breast Pads

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AVENT America, Inc.

1.800.54.AVENT

Exclusively Imported and Distributed by AVENT America, Inc.
Made in England. Philips of Holland. All logos brands and product names are trademarks or registered trademarks of their respective holders and are hereby recognised as such.
Manufactured by AVENT, Suffolk, CO10 7QS, England under Licence from Nipplette™ Ltd.

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PHILIPS

US

The Nipplette™ uses the principles of tissue expansion to stretch and lengthen short breast ducts. Because of the unique valve, the instrument can apply suction for extended intervals. It is through prolonged use that permanent correction is achieved. It is painless and easy to use but medical consultation is recommended.

Indication for use:

The Nipplette may be used to correct flat and inverted breast nipples which developed as a result of hereditary factors or during puberty. Prolonged use will draw the nipple out, cosmetically enhancing the contour of the nipples. The Nipplette has also been proven to be effective for drawing out inverted nipples to facilitate nursing. However, the Nipplette should not be used for permanent correction until lactation is completed.

Contraindication:

The Nipplette should NOT be used without medical advice in the following circumstances:

- when inverted nipples developed later in life
- during the last trimester of pregnancy

Directions for use:

1. Push the tip of the syringe port firmly into the end of the Nipplette opposite the thimble. Be sure the syringe “locks” into place against the spring loaded valve.
2. Hold the Nipplette over the nipple with one hand and apply suction by pulling on the syringe with the other. (A small

smear of a non-petroleum based lubricant at the base of the Nipplette may improve suction).

3. Pull as firmly as is comfortable, increasing suction weekly.
4. When the nipple has been pulled out, hold the valve and pull on the syringe to separate the valve from the syringe. You are now free to continue your daily activities wearing the Nipplette. It is concealed under loose clothing or may be held in place by a brassiere.
5. To remove the Nipplette, push the syringe into the valve to release the vacuum.
6. To obtain the best results, use a comfortable suction and wear the Nipplette for as many hours as comfortable. It is possible to wear the Nipplette overnight.

Correction should be seen within 3-12 weeks of consistent (8 hours/day) daily use. Length of treatment required will vary depending on the duration of daily use and in cases of inversion, the severity of the condition.

When the nipple can easily be drawn to the end of the thimble, use of the Nipplette can be reduced. Some retraction may occur because of the nature of elastic tissue in the nipple. A repeated session of 1-3 weeks will help restore and maintain the nipple shape.

Precaution:

Pulling too hard on the nipple with the Nipplette will cause discomfort. If this occurs lessen the suction.

Do NOT Sterilize the unit.

Wash the Nipplette after each use by pumping warm soapy water through the system, followed by clear water rinsing.

Adverse Reactions:

Excessive suction force applied to the nipple could cause bleeding to occur. Some individuals may notice a small amount of discharge from their nipples, this is entirely normal. If you experience more discharge or pain, decrease the amount of suction and consult a medical professional. A toll free telephone number is available for questions or concerns about usage of the Nipplette.

Call AVENT America, Inc. at 1.800.54.AVENT

**THE Nipplette IS A PERSONAL CARE PRODUCT.
DUE TO HEALTH REASONS, NO REFUNDS OR
EXCHANGES ONCE OPENED.**

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AVENT America, Inc. 1.800.54.AVENT